HOMEDICS®

LEADERS IN BEAUTY & WELLBEING

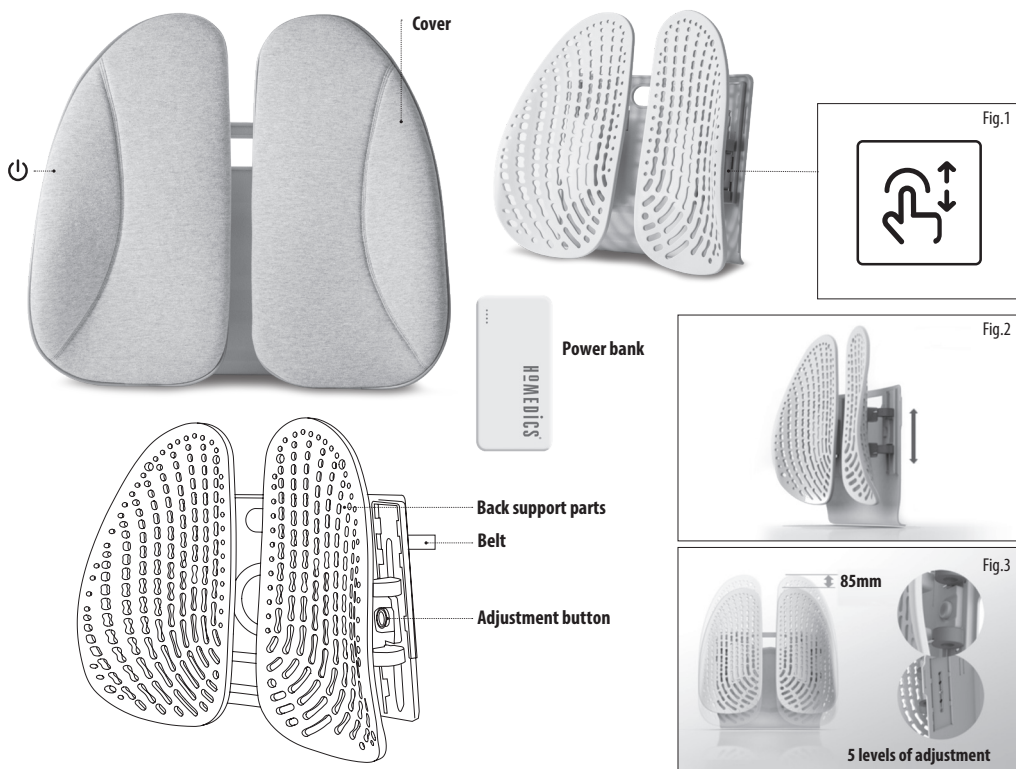
ERGONOMIC BACK SUPPORT WITH COVER & HEAT

3 YEAR GUARANTEE

ER-BS200H

PRODUCT FEATURES:

INSTRUCTIONS FOR USE:

Before using, place the Ergonomic Back Support in a cool and ventilated area for some time, in order to eliminate any residue smell of material. The Ergonomic Back Support is USB rechargeable and can be used with or without the Cover and Heat function.

How to use

1. Place the back support on a chair.
2. Strap the back support securely on the chair by using the strap on the back of the product.
3. Choose the preferred height by pressing the button (Fig. 1) to slide the back support up and down (Fig.2). The Ergonomic Back Support has 5 levels of adjustable height up to 85mm (Fig. 3).

Heat function

To use the ergonomic back support with heat, add the cotton cover to the back support. The heat function does not work without the cotton cover.

1. Connect the 80cm USB cord on the fabric cover to the power bank. A full charge requires approximately 3 hours for 2 hours use.
2. Turn on the power button on the fabric cover to activate the heat function; a red light will switch on. The Ergonomic Back Support can be used while the power bank is plugged in.
3. The Ergonomic Back Support can heat up to 42°C. It takes approximately 3 minutes to heat up to 33°C and approximately 15 minutes to reach 42°C.

4. Turn off the appliance by pressing the power button; the red light will switch off.

Cleaning

Note: This product cannot be washed. If accidentally stained during use, use a dry towel to carefully remove the dirt.

Unplug the appliance and allow to cool before cleaning. Only use a dry cloth to wipe dirt on the surface of the product.

- Never allow water or any other liquids to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

Do not dry clean. Do not wash. Do not bleach.

Not to be used by very young children (0-3 years). Do not insert pins.

Read operator's manual

Storage

Always turn off the appliance first and wait for it to cool down. Place the appliance in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface.

READ ALL INSTRUCTIONS BEFORE USE.
SAVE THESE INSTRUCTIONS FOR FUTURE
REFERENCE.

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- Do not place or store appliance where it can fall or be pulled into a bath or sink. Do not place in or drop into water or other liquid.
- DO NOT reach for an appliance that has fallen in water or other liquids. Switch off at the mains and unplug it immediately. Keep dry – DO NOT operate in wet or moist conditions.
- NEVER insert pins, metallic fasteners or objects into the appliance or any opening.
- Use this appliance for the intended use as described in this booklet. DO NOT use attachments not recommended by HoMedics.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT use outdoors.
- DO NOT crush. Avoid sharp folds.
- DO NOT attempt to repair the appliance. There are no user serviceable parts. All servicing of this appliance must be performed at an authorised HoMedics Service Centre.
- If you have any concerns regarding your health, consult a doctor before using this appliance.
- Use of this product should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your GP.
- Pregnant women, diabetics and individuals with pacemakers should consult a doctor before using this appliance. Not recommended for use by individuals with sensory deficiencies including diabetic neuropathy.
- DO NOT use on an infant, invalid or on a sleeping or unconscious person. DO NOT use on insensitive skin or on a person with poor blood circulation.
- This appliance should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- NEVER use directly on swollen or inflamed areas or skin eruptions.
- Do not use for longer than the recommended time.
- Failure to follow the above may result in the risk of fire or injury.
- This appliance contains batteries that are non-replaceable.
- The battery must be removed from the

- appliance before it is scrapped;
- The appliance must be disconnected from the supply mains when removing the battery;
- The battery is to be disposed of safely.
- Not intended for medical use in hospital
- Examine the appliance frequently

THREE YEAR GUARANTEE

FKA Brands Ltd guarantees this product from defect in material and workmanship for a period of 3 years from the date of purchase, except as noted below. This FKA Brands Ltd product guarantee does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorised accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of FKA Brands Ltd. This guarantee is effective only if the product is purchased and operated in the UK / EU. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and / or authorised, or repair of products damaged by these modifications is not covered under this guarantee. FKA Brands Ltd shall not be responsible for any type of incidental, consequential or special damages.

To obtain guarantee service on your product, return the product post-paid to your local service centre along with your dated sales receipt (as proof of purchase). Upon receipt, FKA Brands Ltd will repair or replace, as appropriate, your product and return it to you, post-paid. Guarantee is solely through HoMedics Service Centre. Service of this product by anyone other than the HoMedics Service Centre voids the guarantee. This guarantee does not affect your statutory rights.

For your local HoMedics Service Centre, go to www.homedics.co.uk/servicecentres

Battery replacement

Your product includes a rechargeable battery designed to last the lifetime of the product. In the unlikely event that you should require a replacement battery, please contact Customer Services, who will supply details of the warranty and out-of-warranty replacement battery service.

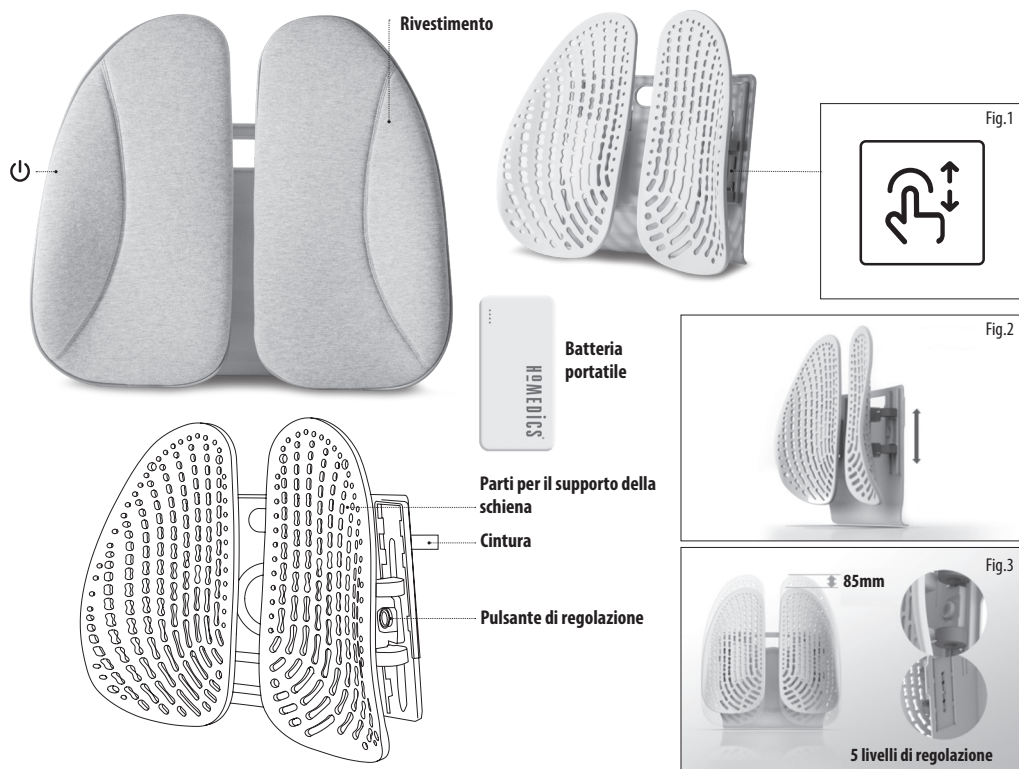
Battery directive

 This symbol indicates that batteries must not be disposed of in the domestic waste as they contain substances which can be damaging to the environment and health. Please dispose of batteries in designated collection points.

WEEE explanation

 This marking indicates that this product should not be disposed of with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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Distributed in the UK by: FKA Brands Ltd, Somerhill Business Park,
EU Importer: FKA Brands Ltd, 29 Earlsfort Terrace, Dublin 2, Ireland