Homedics

**deluxe Shiatsu
foot massager with heat**

✓ Register your product today at
www.homedics.co.uk/product-registration

3 YEAR GUARANTEE

FM-TS9-EU

PRODUCT FEATURES

INSTRUCTIONS FOR USE:

Always ensure the appliance is set to the 'OFF' before inserting the plug into a mains outlet.

1. Plug the appliance into a 220-240V mains outlet and switch on.
2. Place the massager on the floor in front of your chair.
3. To activate the massage, press the 'Power' button once.
4. Position your feet over the rotating massage heads, applying a level of pressure that feels comfortable for you. It is highly recommended that this appliance is used with socked feet.
5. To activate heat with the massage, press the 'Power' button a second time.
6. To switch off, press the 'Power' button till the massage heads stop moving.

NOTE:

- DO NOT attempt to stand on this Foot Massager. Use only while seated.
- The Heat function will only work when the massage is in use.
- DO NOT use the massage or heat functions continuously for more than 15 minutes.

MAINTENANCE:

Cleaning

Unplug the appliance and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.

- Never allow water or any other liquids to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

Storage

Place the appliance in its box or in a safe, dry, cool place.

To avoid breakage, DO NOT wrap the power cord around the appliance. DO NOT hang the unit by the cord.

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FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
FKA Brands Ltd, 29 Earlsfort Terrace, Dublin 2, Ireland