

HOMEDICS®

LEADERS IN HEALTH & WELLBEING



Register your product today at
www.homedics.co.uk/product-registration



COMFORT^{PRO} TRANSFORM

RECHARGEABLE THROW

instruction manual

HCM-T210GYQVC-GB
HCM-T210PKQVC-GB

3 YEAR GUARANTEE

CHARGING THE BATTERY PACK

The battery pack is stored in a small pocket on the inside of the wrap. It can be charged while the product is in use if desired, but the heat function is not available while charging. Alternatively you can disconnect the battery pack from the product to charge it.

To check the battery pack charge at any time – press the button on the side of the pack.

Four blue indicator lights = full charge.

No lights = no charge remaining.





To charge:

1. Ensure that the device is switched off (power button is not lit).
2. Plug the micro USB end of the charging cable provided into the INPUT socket of the battery pack.

3. Plug the USB end of the cable into a USB power socket (not provided).
4. A full charge requires approximately 4-5 hours. The light indicator on the battery pack will flash blue while the unit is charging. When a full charge is reached, the battery indicator lights will all be illuminated. The vibration mode of the massager can be used while plugged in and it will continue to charge. However, heat cannot be used while on charge.
5. Once fully charged, unplug and remove the charging lead from the battery pack. The battery should last for up to 1.5-2 hours of cordless use.

INSTRUCTIONS FOR USE:



1. Locate the USB cord which is attached within the inside pocket of the wrap and plug it into the OUTPUT port on rechargeable battery pack. (Fig. 1) Pop the battery pack back into the pocket of the wrap.
2. Place the wrap over your shoulders. (Fig. 2)
3. To activate the massage action, press the power button  on the control pad. Then press the **M** button to activate the massage on the lowest setting. To increase intensity, press  once for medium and a second time for high intensity. (Fig. 3)
4. To change massage program from steady to pulse, press the **M** button once. Pressing **M** button a second time will turn the vibration off. (Fig. 3)
5. To activate the heat function, press the heat button  on the control pad. To turn off the heat function, simply press the heat button  again. (Fig. 3)

NOTE: You can use the heat without having the vibration.

6. To turn the device off completely, press the power button .

NOTE: The battery pack will automatically turn off after 2 minutes of disuse in order to conserve power. To use the battery pack after it has automatically powered off, simply press the power button again, and operate the wrap as normal.

NOTE: There is a 30 minutes auto shut-off on this appliance for your safety. This feature should not be considered as a substitute for 'OFF'. Always remember to switch the appliance off when not in use.

MAINTENANCE:

Cleaning

Unplug the appliance and allow to cool before cleaning. Clean the surface of the fabric only with a soft, slightly damp sponge.

- Never allow water or any other liquids to come into contact with the battery pack or USB leads.
- Do not immerse in any liquid to clean.
- Never use any chemical cleaning agents.

Storage

Place the appliance in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface.



FKA Brands Ltd

Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
Customer Support: +44(0) 1732 378557 | support@homedics.co.uk

GB-HCMT210XXQVC-0620-01