Homedics

glo body analyser scale

✓ Register your product today at
www.homedics.co.uk/product-registration

3 YEAR GUARANTEE

SCL-BBC200-BK3R

PRODUCT FEATURES:


PREPARING YOUR SCALE:

1. Open the battery compartment on the scale underside.
2. Insert 4 AAA batteries observing the polarity signs (+ and -) inside the battery compartment.
3. Close the battery compartment.
4. Select lb, kg or st lb weight mode by pressing the units button on the underside of the scale.
5. Position scale on a firm flat surface. Allow the reading to set to 0.00 before stepping on the scale. Repeat every time the scale is moved.

HOMEDICS SCALES APP:

The scales can be used with our free to download Homedics Scales app, available from Google Play and the App Store. The user-friendly app allows you to track your weight and provides detailed insight into your body composition, including BMI, bone mass and muscle mass.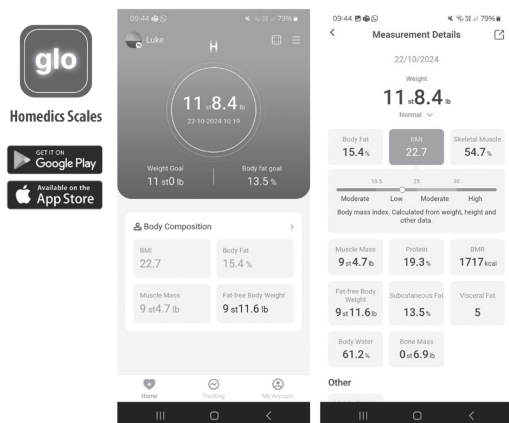

OPERATING YOUR SCALE:

To get the most from your scale please download the app, set up your profile, and connect your device to the scale. Your scale will then display weight, BMI and Body Fat % and these will be saved to the app along with many other readings. Without the app, or if shoes/socks are worn, only a weight reading can be displayed.

1. Step on with bare feet touching each of the four metal contacts. Stand still while the scale computes your readings.
2. The display will flash when your readings have been calculated.
3. Step off. Your weight will be displayed for a few seconds
4. The scale will switch off. Firstly the weight value will turn off, and a few seconds later the night light will turn off – if it is turned on.

MOTION SENSING NIGHT LIGHT:

- The motion sensing night light can be turned on or off using the switch on the underside of the scale. There are 2 levels of brightness – press once to turn on high level, press again for the low level, and press a third time to turn off.
- The light will activate when you are 70cm or closer to the scales. It will remain lit for up to 45secs if you stay within a 70cm radius, including when you are weighing yourself. After this time, it will turn off.
- To save battery power, if you continue to move around the scale for longer than 45 secs the light will switch off and will not activate again until 10 mins have passed.

WARNING INDICATORS:

- 'Lo' is displayed when batteries are low
- 'Err' is displayed when the weight limit is exceeded

ADVICE FOR USE AND CARE:

- Always weigh yourself on the same scale placed on the same floor surface. Do not compare weight readings from one scale to another as some differences will exist due to manufacturing tolerances.
- Placing your scale on a hard, even floor will ensure the greatest accuracy and repeatability.
- Weigh yourself at the same time each day, before meals and without footwear. First thing in the morning is a good time.
- Your scale rounds up or down to the nearest increment. If you weigh yourself twice and get two different readings, your weight lies between the two.
- Clean your scale with a damp cloth. Do not use chemical cleaning agents.
- Do not allow your scale to become saturated with water as this can damage the electronics.
- Treat your scale with care – it is a precision instrument. Do not drop it or jump on it.
- **Caution:** the platform may become slippery when wet.

TROUBLESHOOTING GUIDE:

If you experience any difficulty in the use of your scale:

- Check the battery is correctly fitted.
- Check that you have selected your preference of stones/pounds, kilograms or pounds.
- Check that the scale is on a flat, level floor and not touching against a wall.
- If when you use the scale there is no display at all, replace the batteries.
- Avoid use on carpeted or uneven surfaces.
- If you have particularly dry skin, you may need to moisturize your feet to get a reading through the metal contacts. Alternatively, use after a bath or shower, but always dry feet first to avoid a slip hazard.

READ ALL INSTRUCTIONS BEFORE USE. SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.

- **DO NOT** use if you have either implanted or external medical devices such as a cardiac pacemaker, artificial lung, or wearable electrocardiograph.
- **DO NOT** use while pregnant.
- Keep batteries away from children.
- Keep scale dry and away from water. Do not step onto the glass with wet feet as it will be a slip hazard.
- **DO NOT** jump on the scale, or stand on the outer edge of the platform which could cause it to tip.
- **DO NOT** use the scale if the glass appears cracked or broken.
- **DO NOT** overload the scale beyond its stated capacity.
- This product is intended for personal monitoring only and should not be used for treating or diagnosing any medical conditions.

3 YEAR GUARANTEE

FKA Brands Ltd guarantees this product from defect in material and workmanship for a period of 3 years from the date of purchase, except as noted below. This FKA Brands Ltd product guarantee does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorised accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of FKA Brands Ltd. This guarantee is effective only if the product is purchased and operated in the UK / EU. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and / or authorised, or repair of products damaged by these modifications is not covered under this guarantee. FKA Brands Ltd shall not be responsible for any type of incidental, consequential or special damages.

To obtain guarantee service on your product, return the product post-paid to your local service centre along with your dated sales receipt (as proof of purchase). Upon receipt, FKA Brands Ltd will repair or replace, as appropriate, your product and return it to you, post-paid. Guarantee is solely through HoMedics Service Centre. Service of this product by anyone other than the HoMedics Service Centre voids the guarantee. This guarantee does not affect your statutory rights.

For your local HoMedics Service Centre, go to
www.homedics.co.uk/servicecentres

WEEE explanation

This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

Battery directive

This symbol indicates that batteries must not be disposed of in the domestic waste as they contain substances which can be damaging to the environment and health. Please dispose of batteries in designated collection points.

CAUTION

- Replacement of the battery with an incorrect type can defeat a safeguard.
- Do not dispose of a battery into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion;
- Do not leave a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas
- Do not expose the battery to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas

ELECTROMAGNETIC COMPATIBILITY

This product is intended for domestic use.

ADVICE TO THE USER

This product is a sensitive electronic instrument and as such may temporarily be affected by Radio Transmitting Devices being used in close proximity to it (such as Mobile Phones, Walkie Talkies, CB Radios, Radio Model Controllers, and some Microwave Ovens, etc.) Additionally, extreme levels of Electrostatic Interference may cause this product to temporarily malfunction. In such case it may be necessary to remove and re-install the battery to re-establish normal working. If the problem persists then contact your local agent.

Operating frequency Band: 2402–2480MHz
CE BT Max Transmitted Power: 0.92dBm

Operating frequency band: 5275–5875MHz
5.8G RADAR Max Transmitted Power: -5.62dBm

Hereby, FKA Brands Ltd, declares the radio equipment is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

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FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
FKA Brands Ltd, 29 Earlsfort Terrace, Dublin D02AY28, Ireland