
Homedics

Modulair™ compression complete compression system

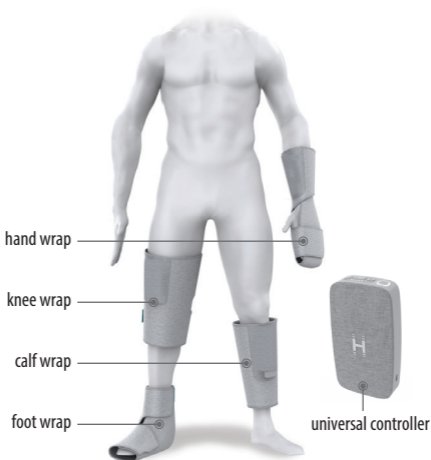
Register your product today at
www.homedics.co.uk/product-registration

3 YEAR GUARANTEE
SR-CM10 Series

PRODUCT FEATURES:

The modula^{ir}™ Controller Module is equipped with our intelligent SMRTflex chip. This technology gives our controller module the ability to detect which modula^{ir}™ compression wrap it's docked into and perform the programmes specific to that wrap. This allows you to add on to your system and build a compression regimen that fits your needs, all while doing it with the same battery module.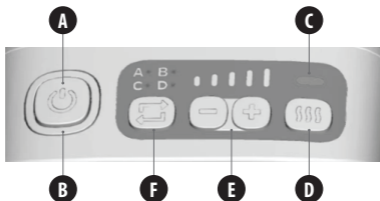

A / POWER  - To turn the massager on/off, press the power button. The ring around the power button will illuminate white to indicate that the unit is on.

NOTE: The massager will begin on mode A and 60% intensity with no heat.

B / BATTERY INDICATOR  - The colour indicates the battery level remaining.

- Solid White: 40% - 100% Charge
- Solid Red: 20 - 40% Charge
- Flashing Red: ≤20% Charge

C / HEAT INDICATOR  - The colour indicates the intensity of the heat selected.

- Green: Low
- Blue: Medium
- Purple: High

D / HEAT BUTTON  - To select between three levels of heat therapy (Low, Medium, and High), press the heat button to reach the desired level.

E / INTENSITY  - Press the intensity button to select your preferred level of pressure. The intensity level is indicated on the LED screen to the left of the button.

- 20%
- 40%
- 60%
- 80%
- 100%

F / MODE BUTTON  - Press the mode button to select between the four massage modes. The mode is indicated on the LED screen directly above the button.

- (A) Alternating
- (B) Full
- (C) Sequential
- (D) Pulsing

INSTRUCTIONS FOR USE:

Congratulations on your purchase of a Homedics modular™ product. Modular™ is the first pneumatic compression system that allows you to customize your experience by offering compression wraps and products for all parts of the body all using the same controller module for complete flexibility. To start using your modular™ system, you'll need at least one modular™ controller module (Model SR-CMX10) and at least one modular™ compression wrap.

1. Your modular™ compression system uses a modular controller module (SR-CMX10). To start, when opened, your controller module may require charging. To charge the battery, plug one end of the USB cord into the unit and plug the other end into an adapter. Your controller module will be fully charged after a maximum of 5 hours charge time, and the charge will last up to 5 hours.


NOTE: This device only comes with a USB cord, not an adapter. A USB charger will also charge the units. Do not attempt to use a cord that is not a USB-type cord. Use only with Listed/Certified ITE Power Supply. The units will not operate when charging. Use a 5V/2A (or 5V/1A, slower) charger, but not higher than 5V.

2. Your compression wrap can be used on either the left or right side.
3. Use the fastening tape to adjust the wrap so it's snug but comfortable. Do not overtighten.
4. Connect your controller module to the compression wrap, sliding it into the dock until it "clicks" into place. See image to the right.
5. Press the power button to turn on the unit. The display will illuminate to confirm the unit is on.
6. Press the "MODE" button to select between the 4 massage modes. Press the "HEAT" button to toggle between 3 levels of heat. Press the "INTENSITY" button to adjust the intensity of the air compression.
7. The unit will turn off automatically after 15 minutes.


CAUTION: To avoid injury, always remain seated when using massager.

4 MASSAGE MODES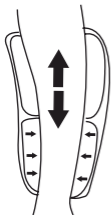


MODE A: ALTERNATING

The first airbag is inflated to the desired pressure and maintains the pressure for 5 seconds. The air is then released for 2 seconds. The second airbag begins to inflate to the desired pressure and maintains the desired pressure for 5 seconds. The air is then released for 2 seconds. Both airbags inflate simultaneously and maintain pressure for 5 seconds. The pattern continues to repeat until auto shutoff at 15 minutes.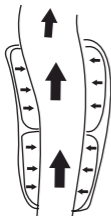


MODE B: FULL

The two airbags are inflated to the desired pressure at the same time. The airbags hold the pressure for 5 seconds and then release the air for 3 seconds. This pattern continues to repeat until auto shutoff at 15 minutes.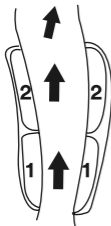

MODE C: SEQUENTIAL

The lower airbag is inflated to the desired pressure, and the pressure is held for 3 seconds. The upper airbag then inflates to the desired pressure. Air pressure is held in both airbags for 5 seconds and then released for 5 seconds. This pattern continues to repeat until auto shutoff at 15 minutes.

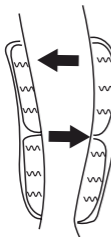MODE D: PULSING

The two airbags are inflated to the desired pressure at the same time, then the airbags immediately release the air for 1 second. The airbags then continue to inflate to the desired pressure. This pattern continues to repeat until auto shutoff at 15 minutes.

MAINTENANCE:

Cleaning

- Disconnect the battery from the compression wrap and allow to cool before cleaning.
- Clean only with a soft, slightly damp sponge.
- Never allow water or any other liquids to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

Storage

Allow the product to cool down and then place in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, DO NOT wrap the USB cable around the appliance. DO NOT hang the unit by the USB cable.

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING: READ ALL INSTRUCTIONS BEFORE USING.

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for a unit that has fallen into water. Unplug it immediately.

- NEVER use pins or other metallic fasteners with this appliance.

WARNING – TO REDUCE THE RISK OF BURNS, FIRE, ELECTRIC SHOCK, OR INJURY TO PERSONS:

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the

- hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
 - Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
 - DO NOT use outdoors.
 - Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by Homedics; specifically, any attachments not provided with the unit.
 - DO NOT carry this appliance by the supply cord or use cord as a handle.
 - NEVER operate this appliance if it has a damaged USB cable, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water. Return the appliance to the HoMedics Service Centre for examination and repair.
 - Keep the cord away from heated surfaces.
 - NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
 - NEVER drop or insert any object into any opening.
 - DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
 - DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.

- To disconnect, turn all controls to the off position, then remove control/battery unit.
- DO NOT stand on or in appliance. Use only while seated.
- Keep long hair away from massager while in use.
- DO NOT use massager in close proximity to loose clothing or jewellery.
- Never operate on a soft surface such as a bed or couch where the air openings may be blocked.
- Prevent unintentional starting. Ensure the switch is in the off position before connecting to battery pack, picking up or carrying the appliance. Carrying the appliance with your finger on the switch or energizing appliance that have the switch on invites accidents.
- Disconnect the battery pack from the appliance before making any adjustments, changing accessories, or storing appliance. Such preventive safety measures reduce the risk of starting the appliance accidentally.
- Use heated surfaces carefully. May cause serious burns. DO NOT use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- Recharge only with the USB cable specified by the manufacturer. A USB cable that is suitable for one type of battery pack may create a risk of fire when used with another battery pack.
- Use appliances only with specifically designated battery packs. Use of any other battery packs may create a risk of injury and fire.
- When battery pack is not in use, keep it away from other metal objects, like paper clips, coins, keys, nails, screws or other small

metal objects, that can make a connection from one terminal to another. Shorting the battery terminals together may cause burns or a fire.

- Under abusive conditions, liquid may be ejected from the battery; avoid contact. If contact accidentally occurs, flush with water. If liquid contacts eyes, seek medical help.
- Liquid ejected from the battery may cause irritation or burns.
- DO NOT use a battery pack or appliance that is damaged or modified. Damaged or modified batteries may exhibit unpredictable behavior resulting in fire, explosion or risk of injury.
- DO NOT expose a battery pack or appliance to fire or excessive temperature. Exposure to fire or temperature above 130°C may cause explosion.
- Follow all charging instructions and do not charge the battery pack

or appliance outside of the temperature range specified in the instructions. Charging improperly or at temperatures outside of the specified range may damage the battery and increase the risk of fire. Product operating and charging range: 0°C – 40°C

- This appliance contains batteries that are only replaceable by skilled persons.
- This appliance contains batteries that are non-replaceable.

SAVE THESE INSTRUCTIONS

CAUTION – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- Consult your doctor prior to using this product, if:
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- NOT recommended for use by diabetics.
- NEVER leave the unit

- unattended, especially if children are present.
- NEVER cover the unit when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massager has a stimulating effect and can delay sleep.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- NOT for use by children.
- This appliance is intended for household use only.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies.

NOTE: This unit will shut off automatically after 15 minutes of use. If your unit overheats, turn off the unit and let cool for 30 minutes before turning on again and continuing use.

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3 YEAR GUARANTEE

FKA Brands Ltd guarantees this product from defect in material and workmanship for a period of 3 years from the date of purchase, except as noted below. This FKA Brands Ltd product guarantee does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorised accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of FKA Brands Ltd. This guarantee is effective only if the product is purchased and operated in the UK / EU. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and / or authorised, or repair of products damaged by these modifications is not covered under this guarantee. FKA Brands Ltd shall not be responsible for any type of incidental, consequential or special damages.

To obtain guarantee service on your product, return the product post-paid to your local service centre along with your dated sales receipt (as proof of purchase). Upon receipt, FKA Brands Ltd will repair or replace, as appropriate, your product and return it to you, post-paid. Guarantee is solely through HoMedics Service Centre. Service of this product by anyone other than the HoMedics Service Centre voids the guarantee. This guarantee does not affect your statutory rights.

For your local HoMedics Service Centre, go to
www.homedics.co.uk/servicecentres

WEEE explanation

 This marking indicates that this product should not be disposed with other household wastes throughout the EU.

To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
FKA Brands Ltd, 29 Earlsfort Terrace, Dublin D02AY28, Ireland