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Luxury Foot Massager

with Heat

**Instruction Manual and  
Warranty Information**

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**2** year  
limited warranty

**IMPORTANT SAFEGUARDS**  
WHEN USING ELECTRICAL PRODUCTS,  
ESPECIALLY WHEN CHILDREN ARE  
PRESENT, BASIC SAFETY PRECAUTIONS  
SHOULD ALWAYS BE FOLLOWED,  
INCLUDING THE FOLLOWING:  
READ ALL INSTRUCTIONS BEFORE USING.

**DANGER-** TO REDUCE THE RISK OF  
ELECTROCUTION:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- Do not reach for an appliance that has fallen into water. Unplug it immediately.
- Do not use while bathing or showering.
- Do not place or store appliance where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid.

**WARNING -** TO REDUCE THE RISK OF  
BURNS, ELECTROCUTION, FIRE OR INJURY  
TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.

**Caution:**

**In case of pregnancy, diabetes or illness, consult your doctor before using this massager.**

**Caution:**

**All servicing of this massager must be performed by authorized HoMedics Service Personnel only.**

- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination or repair.
- Keep cord away from heated surfaces.
- Never block the air openings of the appliance or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep air openings free of lint, hair, etc.
- Never use while sleeping or fall asleep while using the massager.
- Never drop or insert any object into any opening.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- Do not carry this appliance by supply cord or use cord as handle.
- To disconnect, depress all control buttons to the "OFF" position, then remove plug from outlet.
- This appliance is designed for household use only.
- Use heated surfaces carefully. May cause serious burns. Do not use on sensitive areas.

**SAVE THESE INSTRUCTIONS**

## Caution: - Please read all instructions carefully before operating.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- If you have any concerns regarding your health, consult your doctor before using the Foot Revitalizer.
- Individuals with pacemakers should not use this product.
- If you experience pain in a muscle or joint for a prolonged period of time, discontinue use and consult your doctor. Persistent pain could be a symptom of a more serious condition.
- Never use Foot Revitalizer directly on open wounds or discolored areas.
- Do not use for more than 15 minutes at a time. Excessive use could cause the product to overheat. Should this occur, discontinue use and allow the unit to cool before operating.
- If heat function is left on for a prolonged period of time, lightly touch bulb surface to ensure the heat level is comfortable. If it is too hot, turn heat function off, allowing surface to cool.
- Never stand or bear heavy weight on the unit. Unit must be used while seated only. Standing or bearing heavy weight upon the unit will cause irreparable damage.
- Never try to open or stick objects into any part of the Foot Revitalizer. Damages of this nature are not covered under the HoMedics warranty.
- The massage unit must be in the "OFF" position before plugging into the wall outlet. (Massage button is not depressed.)
- Use this appliance for its intended use as described in this manual.
- This unit is not a toy. Children should not use it or play with it.
- Never use while sleeping.
- Do not place or drop into water or any other liquid.

## Instructions for Use

To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

1. It is highly recommended that Foot Revitalizer is used with socked feet. Do not use the unit while wearing shoes.
2. Place Foot Revitalizer on the floor in front of your chair or seat. Make sure the unit is in the "OFF" position. Buttons are located on the bottom of the unit.
3. Plug the unit into the power source (120V).
4. To activate massage action, press power button once. Blue LED's will illuminate.
5. The second press of the button will activate the heat feature. Red LED's will illuminate to confirm your selection.

This appliance is intended for household use only.

**NOTE: Do not use the heat if you have sensitive skin, poor circulation, discolored areas, swelling, burning, inflammation or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician. In case of pregnancy, diabetes or illness, consult your doctor before using massager.**

# Foot Revitalizer

Luxury Foot Massager

*with Heat*

## Maintenance

**Caution:** All servicing of this unit must be performed by authorized HoMedics service personnel only. This appliance has no user serviceable parts.

**To Clean** Unplug the unit and allow it to cool before cleaning. Clean only with a soft, damp cloth. Never allow water or any other liquid to come into contact with the control switch.

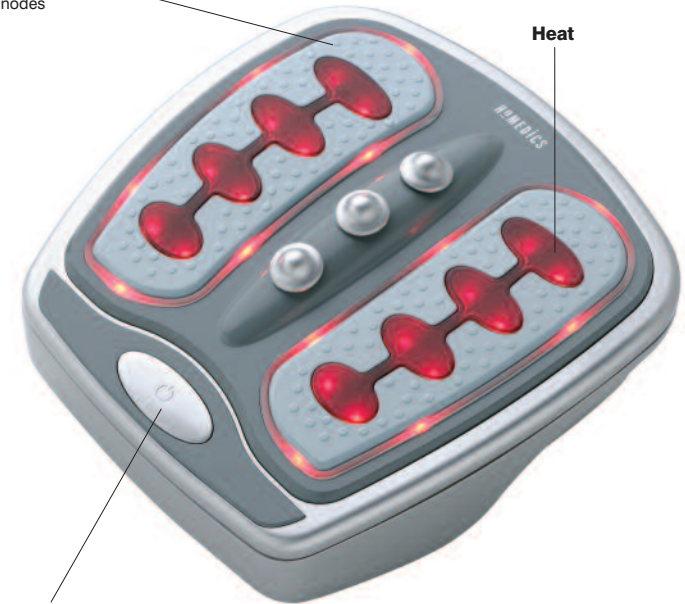
- After cleaning, wipe surfaces with a dry cloth.
- Never use abrasives, detergents or cleaning agents.
- Periodically straighten cord if it becomes twisted.

**To Store** Unplug the appliance from the outlet and allow to cool.

- Store in its box or a clean, dry place.
- To avoid breakage, do not wrap the power cord around the unit.
- Wrap cord around the storage posts located on the bottom of the unit.
- Do not hang the unit by the power cord.

**Massage Platform**  
With raised nodes

**Heat**

**Massage On/Off Button**  
**Heat On/Off Button**

## LIMITED TWO YEAR WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of two years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Consumer Relations  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI 48390

No COD's will be accepted.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HoMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HoMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resaled products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

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