

HOMEDICS®

LEADERS IN HEALTH & WELLBEING



Register your product today at
www.homedics.co.uk/product-registration



CONVERTIBLE PILLOW & FOOT MASSAGER WITH HEAT

instruction manual

HCM-70H-2-GB

3 YEAR guarantee

PRODUCT FEATURES:

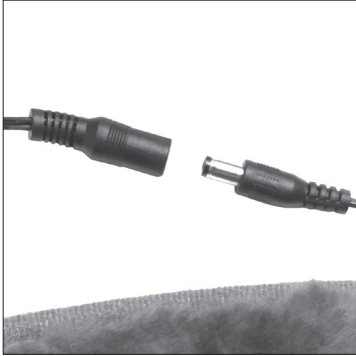


Fig. 1



Fig. 2

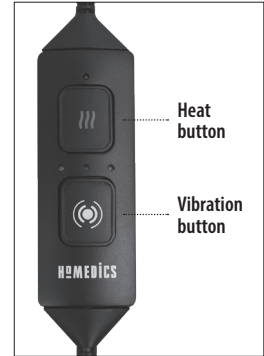


Fig. 3

INSTRUCTIONS FOR USE

1. Insert the jack into the socket of the massager (Fig. 1)
2. Plug the adaptor into a AC 220-240v mains outlet.
3. Place your feet inside the massager (Fig. 2).
4. To activate the massage action, press the vibration button on the remote control. The massage will activate on the low setting in steady vibration program (Fig. 3)
5. To change massage intensity, press the vibration button again.
NOTE: there are 3 different intensities, low, medium and high. To change mode, press the vibration button.
6. To activate the soothing heat function, press the heat button on the control pad. The red LED will turn on. To turn off the soothing heat function, simply press the heat button again.
NOTE: You can use the heat without having the vibration.
7. Keep pressing the vibration button until the unit turns off. This will switch off the massage.
NOTE: This will not switch off the heat function, so remember to turn off the heat function too.

NOTE: There is a 15 minute auto shut-off on this appliance for your safety. This feature should not be considered as a substitute for 'OFF'. Always remember to switch the appliance off when not in use.

MAINTENANCE:

Cleaning

Unplug the appliance and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.

- Never allow water or any other liquids to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

Storage

Place the appliance in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, DO NOT wrap the power cord around the appliance. DO NOT hang the unit by the cord.

FKA Brands Ltd

Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
Customer Support: +44(0) 1732 378557 | support@homedics.co.uk



IB-HCM70H2GB-0718-01