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Extendable Percussion Massager

WITH
HEAT

Instruction Manual and Warranty Information

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2 year
limited warranty

IMPORTANT SAFEGUARDS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING **DANGER**—TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- **DO NOT** reach for an appliance that has fallen into water. Unplug it immediately.
- **DO NOT** use while bathing or in shower.
- **DO NOT** place or store appliance where it can fall or be pulled into a tub or sink.
- **DO NOT** place in or drop into water or other liquid.

WARNING—TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.

Caution:
All servicing of this massager must be performed by authorized HoMedics service personnel only.

- **NEVER** operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.
- **NEVER** drop or insert any object into any opening.
- **NEVER** operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- **NEVER** operate on a soft surface such as a couch or bed where the air openings may be blocked.
- Use heated surfaces carefully. May cause serious burns. Do not use over insensitve skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- **DO NOT** operate where aerosol (spray) products are being used or where oxygen is being administered.
- **DO NOT** operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- **DO NOT** carry this appliance by the power cord or use cord as handle.
- To disconnect, turn all controls to the "off" position, then remove plug from outlet.
- **DO NOT** use outdoors.

SAVE THESE INSTRUCTIONS

Caution—Please read all instructions carefully before operating.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product, if
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- Not recommended for use by Diabetics.

- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This unit should not be used by children or invalids without adult supervision.
- This product is not for use in automobiles.

Instructions for Use

- This appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.
1. Always make sure the unit is set to the "OFF" position before inserting the plug into a 120 volt electrical outlet.
 2. To turn the unit on, simply slide the switch located along the top of the units handle up one notch. To increase the intensity of the massage, slide the switch up another notch. To turn the unit off, return the switch to its lowest position. (See page 7)
 3. Turn massage on. To activate heat function, press the heat button located below

the slide switch. To cancel heat function, press the heat button a second time.

NOTE: To eliminate the risk of being burned, never use the heat function without the attachments. Do not use the heat continuously for more than 15 minutes. Do not use the heat if you have sensitive skin, poor circulation, discolored areas, swelling, burned, inflamed or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician. To use heat attachments, locate red massage heads. Turn black massage heads counter-clockwise to remove. Place red nodes on massage and turn clockwise to secure.

4. Apply massager head lightly to the area you wish to massage.
5. Move the unit slowly, holding on the targeted area. Massage the area for a short period of time and move to another area. Never massage the same spot longer than 3 minutes. CAUTION - as stated in the "IMPORTANT SAFETY INSTRUCTIONS" section of this manual, never use this appliance on or near eyes

or other highly sensitive areas.

6. **Handle Extension Instruction:** To utilize the 3" handle extension, slide the locking switch to the unlock position (Fig. 1). Gently extend the head of the unit to the desired position. Slide the locking switch to the lock position (Fig.1).
7. Operation of longer than 15 minutes is not recommended and may cause overheating.
8. When massage is completed, turn unit off, unplug the power supply, allow to cool and store properly in a dry place.

Caution:
All servicing of this unit must be performed by authorized HoMedics service personnel only.

Maintenance

TO CLEAN:

Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe. Never allow water or any liquids to come into contact with the handle.

TO STORE:

Unplug the appliance from the outlet, retract extendable portion and allow it to cool before storing in its box or a clean, dry place. Coil the cord and secure it with the supplied fastener. Do not hang the unit by the powercord.

Figure 1

LIMITED TWO YEAR WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of two years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Service Center
Dept. 168, Suite 3
43155 West Nine Mile Road
Novi, MI 48375

No COD's will be accepted.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

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HHP-300
HHP-300W

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Mail To:
HoMedics Service Center
Dept. 168, Suite 3
43155 West Nine Mile Road
Novi, MI 48375

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