HOMEDICS®

LEADERS IN HEALTH & WELLBEING

✓ Register your product today at
www.homedicsgroup.com/register

TONING PADS

.....
manuel d'instructions

HST-100-EU

3 YEAR guarantee

READ ALL INSTRUCTIONS BEFORE USE.
SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.

IMPORTANT SAFEGUARDS:

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
Children should be supervised to ensure that they do not play with the appliance.
- DO NOT reach for an appliance that has fallen in water or other liquids. Keep dry – DO NOT operate in wet or moist conditions.
- NEVER insert pins or other metallic fasteners into the appliance.
- Use this appliance for the intended use as described in this booklet. DO NOT use attachments not recommended by HoMedics.
- NEVER operate the appliance if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to the HoMedics Service Centre for examination, repair or replacement.
- Keep cord away from heated surfaces.
- Excessive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by the cord or use cord as a handle.
- DO NOT crush.
- DO NOT attempt to repair the appliance. There are no user serviceable parts. For service, send to a HoMedics Service Centre. All servicing of this appliance must be performed by authorised HoMedics service personnel only.
- Do not place or store appliance where it can fall or be pulled into a bath or sink. Do not place in or drop into water or other liquid.
- Failure to follow the above may result in the risk of fire or injury.
- Burns can result from improper use.

SAFETY CAUTIONS :

PLEASE READ THIS SECTION CAREFULLY BEFORE USING THE APPLIANCE.

- **If you have any concerns regarding your health, consult a doctor before using this appliance.**
- Individuals with pacemakers and pregnant women should consult a doctor before using this appliance.
Not recommended for use by Diabetics
- DO NOT use on an infant, invalid or on a sleeping or unconscious person. DO NOT use on insensitive skin or on a person with poor blood circulation.
- This appliance should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- If you feel any discomfort whilst using this appliance, discontinue use and consult your doctor.
- This is a non-professional appliance, designed for personal use and intended to provide gentle muscle stimulation. DO NOT use as a substitute for medical attention.
- DO NOT use before going to bed. The unit has a stimulating effect and can delay sleep.
- Do not use for longer than the recommended time.
- Care should be taken when using heated surfaces. If the product feels excessively hot, switch off and contact the HoMedics Service Centre.
- Long term effects of electrical stimulation are not known.
- Do not use if pregnant.
- Do not use after childbirth. Wait at least six weeks after the birth of your baby (you must consult your doctor first).
- Do not use during a period.
- Do not use if you suffer from heart problems.
- Do not use if you have suspected or diagnosed epilepsy.
- Do not use if you have Multiple Sclerosis.
- Do not stimulate over the carotid sinus nerves (or any part of the front of the throat) especially in persons with known sensitivity to the carotid sinus reflex.
- Do not place over the front of the neck or mouth.
- Do not use on the head.
- Do not apply current through the chest.
- Do not use over swollen, infected or inflamed areas or skin eruptions.
- Keep out of reach of children.
- Do not use over recent scar tissue.
- Do not use following recent surgery.
- Do not use where sensory nerve damage is present by a loss of normal skin. Some persons may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium used.
- The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.

Adverse side effects

Skin irritation and burns beneath the electrodes has been reported with the use of electrical muscle stimulation.

This is not a medical device

This unit is intended to be used only for general physical conditioning. No medical claims are implied or warranted by the use of this product.

MAINTENANCE:

Cleaning

Unplug the appliance and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.

- Never allow water or any other liquids to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

Storage

Place the appliance in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, DO NOT wrap the power cord around the appliance. DO NOT hang the unit by the cord.

WEEE explanation

This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

Battery Directive

This symbol indicates that batteries must not be disposed of in the domestic waste as they contain substances which can be damaging to the environment and health. Please dispose of batteries in designated collection points. This product uses Cel Ni-MH 2500mAh 2.4V rechargeable batteries. When the product reaches the end of its life, please remove the batteries from the product and dispose of them as directed above.

TROUBLESHOOTING

Problem	Possible Cause	Solution
NO DISPLAY SHOWN.	- Battery not installed - Battery installed incorrectly - Battery is dead	- Install Battery - Check polarity of the battery is it in right position - Replace battery
WITH DISPLAY BUT NO FUNCTION.	- No skin contact - Unit not snapped on gel pad correctly - Intensity level at 0 - Microprocessor hang up - Battery is low	- Use water based gel on conductive area and place pad directly onto skin - Check that unit is connected firmly with both connectors to the pad - Push ON/HI button to increase intensity - Reset by removing battery for 1 minute, re-insert battery - Change battery
LCD DISPLAY IN NORMAL BUT NOTHING OCCURS.	- Intensity set to low level - Not enough conductive gel on pad - Used gel is not water-based - Pad is worn or has oil deposits - Battery is low	- Push ON/HI button several times to increase intensity - Put more water based gel between pad and skin - Read label on the gel that you use. Make sure you find "water" or "aqua" on it. - Clean pad or replace (see Hint below) - Change Battery
ALL BUTTONS DO NOT WORK WELL.	- Microprocessor hang up - Battery is low	- Reset by removing battery for 1 minute, re-insert. - Change battery

PRODUCT FEATURES:

- 1 x jumbo display electronic device
- 4 x electrode gel pads
- 1 x connection wire with 4 snap buttons
- 2 x AAA batteries

Toning Programs

Sculpt - Short, deep bursts for effective toning of the upper arms.

Wave - Replicates crunches, ideal for toning the stomach.

Ripple - High intensity for shaping thighs and bottom.

Pulse - Pulsing sensation ideal for problem areas around the hips and waist.

Auto 1/2 - Pre-programmed toning functions, an all over toning solution for all areas.

Function


1. ON/HI: Turns the unit on and increases the output intensity.
2. OFF/LO: Each pressing of this button decreases the intensity by one step down to 10 levels. Turns the unit off.
3. Mode: This button is used to select one of the 6 pre-programmed massage programs. The sport EMS toning device has the 6 programs for selection
4. Time: To set the time for the treatment. Available from 5/10/15/20/25/30 minutes.

Operation

1. Connect the electric wire to the output jack of the device
2. Connect the electric wire to the electrode gel pad with the snap button.
3. Remove the protecting film of the gel pad and stick the gel pad on the body area where you want to have treatment. (Remember to keep the protecting film for storage of the gel pad)
4. Push "ON/HI" button to turn on the device. To switch the device off, press the "OFF/LO" button.
5. Press the "MODE" button repeatedly to select preferred programme. There are 6 different programs for selection - Sculpt, Wave, Ripple, Pulse, Auto_I and Auto_II. Once you selected the mode that you want, an arrow will point to the mode that you have selected.
6. Push the "ON/HI" or "OFF/LO" button to carefully increase or decrease the intensity. The LCD display will show the intensity level which you have selected.
7. Press "TIME" button to set the timer for auto-off, which available 5/10/15/20/25/30 minutes

Hints

You can only adjust the mode and level when the device is on, connected to the gel pad and are in contact with the skin.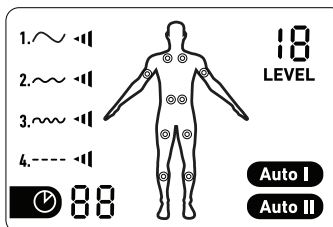

Auto Functions

If the gel pad is removed during use, the intensity is automatically reset to "1". If the gel pad is not placed back into contact with the skin, the unit will turn off after 30 seconds. The unit has an auto-timer. It will shut off after 10 minutes of massage. If you change the massage mode within the 10 minutes, the timer will count-down from the original leftover time.

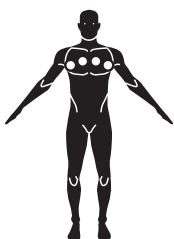
Chart 1
Chest/Pectorials

Chart 2
Abdominals

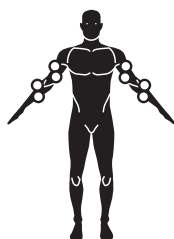
Chart 3
Biceps/Forearm

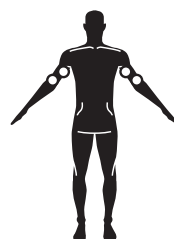
Chart 4
Triceps

Chart 5
Quadriceps

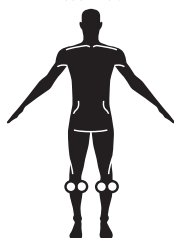
Chart 6
Calves

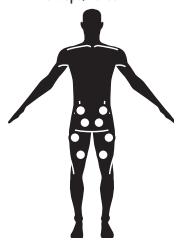
Chart 7
Gluteals/Hamstrings

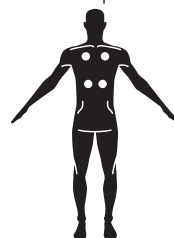
Chart 8
Trapezius/Latissimus dorsi

Toning suggestions:

1. Use your HoMedics sport toning system 5 times a week.
2. Each session should be no longer than 30 minutes
3. Use the strongest training intensity you can - the harder you work your muscles, the better the results you will see, but remember, stimulation should never be uncomfortable.
4. Use your HoMedics sport toning system as part of a normal, healthy lifestyle. Improve your diet and try to increase the amount of exercise you do

Note on batteries

- To prevent the risk of leakage or explosions, never recharge the batteries, apply heat or take them apart.
- If the product is not being used for long periods of time, remove the battery to prevent drain or leakage.
- If liquid leaks from the batteries, throw them away. Thoroughly clean the battery compartment with a dry cloth.

TECHNICAL SPECIFICATIONS

Output Current	23mA (up to 10k ohms load)
Pulse Rate	4Hz ,27Hz
Pulse Duration (Width)	200uS
Timer	5,10,15,20,25 and 30 minutes depending on your selection
Waveform	DC single phasic
Maximum Charge Per Pulse	200uS(Máx)

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