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**Instruction Manual and  
Warranty Information**

MCS-750H

**2** year  
limited warranty

## IMPORTANT SAFEGUARDS

---

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

### DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- ALWAYS unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.

### WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is

**Caution:**  
**All servicing of this massager must be performed by authorized HoMedics service personnel only.**

being administered.

- DO NOT operate under a blanket or pillow. NEVER COVER this appliance when it is in operation. Excessive heating can occur and cause fire, electric shock or injury to persons.
- NOT for use by children.
- DO NOT use this massager in close proximity to loose clothing, jewelry or long hair.
- DO NOT carry this appliance by the power cord or use cord as handle.
- To disconnect, turn all controls to the “off” position, then remove plug from outlet.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair and the like.
- Use heated surfaces carefully. May cause serious burns. DO NOT use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- This appliance is designed for personal, non-professional use.

## SAVE THESE INSTRUCTIONS

---

**CAUTION—PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.**

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product, if
  - You are pregnant
  - You have a pacemaker
  - You have any concerns regarding your health
- NOT recommended for use by Diabetics.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating

**Caution:**  
**To avoid pinching do not lean on the back massage mechanism in the cushion when adjusting your body position.**

**Do not jam or force any part of your body in the moving massage mechanism.**

**Note:**  
**There is a 15-minute auto shut off on the unit for your safety.**

**Caution:**  
**DO NOT use spot Shiatsu with heat for the back in one area for more than 3 minutes.**

and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.

- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- NEVER use this product in automobiles.
- This appliance is intended for household use only.

## Instructions for Use

**Perfect Fit™ technology allows you to set the massager range to the height that is perfect for your back.**

The Quad Shiatsu Massage Cushion comes with a screw located on the back of the unit to protect the massage mechanism during shipment, which must be removed before the first use to allow the massage feature to function. Use the provided Allen wrench. Then, properly dispose of the screw.

**Warning: Failure to remove screw may cause permanent damage to the massage cushion.**

To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

1. Un-tie cord strap, extend cord and insert plug into 120-volt household electrical outlet to power unit.

**Please note:**  
**Not recommended for use on wooden surfaces as the zipper may damage the wood. Caution is also recommended when using on upholstered furniture.**

**NOTE: Only gentle force should be exerted against the unit in order to eliminate any/all risk of injury. You may soften massage force by placing a towel between yourself and the unit.**

2. Attach the cushion to most chairs for a luxurious massage as you read, rest or even work. See pages 8–9 for instructions on the use of the hand control.

## Strapping System

The Massage Cushion incorporates a unique strapping system which allows you to fasten it to almost any chair. Just slip the straps over the seat or chair and adjust fastening straps to secure. Your massager won't slip or slide away.

## Maintenance

### To Store

Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit.

### To Clean

Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- DO NOT immerse in any liquid to clean.
- NEVER use abrasive cleaners, brushes, gasoline, kerosene, glass / furniture polish or paint thinner to clean.
- DO NOT attempt to repair the Massage Cushion. There are no user serviceable parts. For service, call the Consumer Relations telephone number listed in the warranty section.

## FCC Statement

---

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

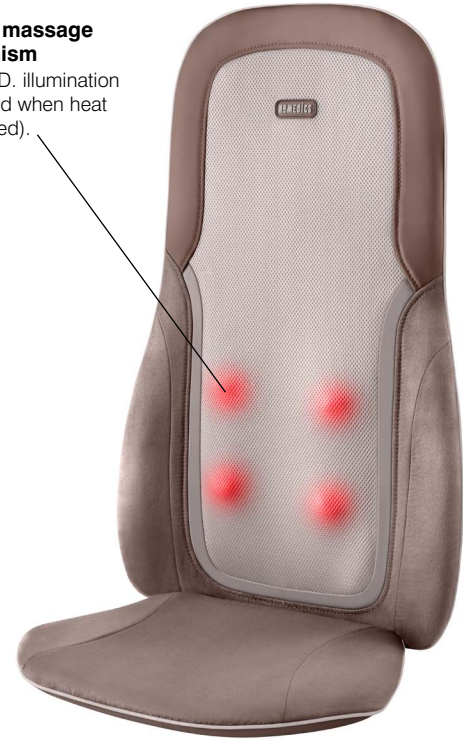
**NOTE:** The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### Moving massage mechanism

With L.E.D. illumination (glows red when heat is activated).

The actual product materials and fabrics may vary from the picture shown.

# Quad Shiatsu Massage Cushion with heat

**NOTE: The moving massage mechanism always “parks” or finishes in its lowest position. It will continue to this position after the power has been turned off. If the electrical supply is interrupted, when power is restored, the mechanism will “park” or move to its lowest position.**

## Demo Button

For a brief demonstration of the features of the Massage Cushion, press the Demo button. The L.E.D. light will blink to indicate active function. The Demo can only be activated when the back mechanism is at the bottom position or Power Button is on. If the back mechanism does not start at the bottom, it will go to the bottom automatically. The unit will briefly go through each function. Once completed, the unit automatically turns off.

## Upper

Upper can only be activated when either Knead, Rolling, or Tapping is on. Press “Upper” once, the LED light will illuminate, Kneading, Rolling, or Tapping the upper back. If “Upper” is already on, press it again, light will stay on the upper back area.

## Lower

Lower can only be activated when Kneading, Rolling, or Tapping is on. Press Lower once, and LED light will illuminate, Kneading, Rolling or Tapping the lower back. If Lower is already on, pressing it again will keep massage on the lower back.

## Kneading Massage

Kneading is a deep penetrating massage. “Knead” can only be activated when Power is on. Press “Knead” once, light on, rolling will be off, activate Knead; press it again, Knead off.

## Spot Massage Feature

While the unit is in Kneading or Tapping, press spot button and either up or down and massager will stop in fixed position. Once the feature is operating, you may adjust the mechanism position by holding down either button until the desired position is reached. The Up arrow will move the mechanism upwards, the Down arrow will move it downward. Once the button is released, the mechanism will stop in that place. Spot position will move from its current position when Upper, Lower, or Perfect Fit is pressed. Press Rolling to turn off the spot and start rolling or Perfect Fit by default.

## Rolling Massage

Rolling is like two hands working slowly up and down your back. “Rolling” can only be activated when Power is on. Press “Rolling” once and LED light will illuminate; Kneading and Tapping will be off. press it again, Rolling will turn off.

## Power Button

To turn on the Massage functions, first press the POWER button. The L.E.D indicator will illuminate and confirm you selection. Knead and Perfect Fit will turn on by default, kneading the full back. To turn off the massage functions, simply press the button again. The L.E.D indicator will blink while the massage mechanism returns to its lowest point and then turn off.

## Perfect Fit™

Perfect Fit™ technology allows you to set the massage range to the height that is perfect for your back. Perfect Fit can only be activated when Power is on. Press Perfect Fit once, LED light will illuminate, and full back massage will turn on. Press twice, the LED light will flash, and the mechanism’s current position is considered the highest distance, mechanism will travel within the designated height and the bottom. Press button a third time, and massage function will return back to full back massage. If the designated height is shorter than 1/3 of the travel distance, it will take 1/3 of the travel distance as the highest height.

## Tapping

Tapping is a fast, invigorating massage. Tapping can only be activated when Power is on. Press “Tapping” once, light on, Rolling will be off, activate high & Tapping; press it twice, activate low & Tapping; press it third time, Tapping off.

## Width Adjustment Feature

Width can only be activated when Rolling or Tapping is on. Press the button to adjust the massage nodes width. With each press, the massage heads will move to the next available position.

## Heat Button

For soothing heat, when either Kneading, Rolling or Tapping massage is on, simply press the heat button and the corresponding red L.E.D light will illuminate. To deselect, press the button again and the corresponding L.E.D. light will turn off.

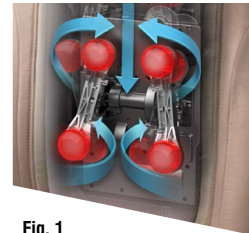
Fig. 1

**NOTE: There is a 15-minute auto shut off on the unit of your safety.**

Distributed by  
**HOMEDICS**

## Limited Two Year Warranty

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of two years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, contact a consumer relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

**This warranty is effective only if the product is purchased and operated in the country in which the product is purchased.** A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com). For Canada, please visit: [www.homedics.ca](http://www.homedics.ca)

HoMedics is a registered trademark of HoMedics, LLC.  
© 2014 HoMedics, LLC. All rights reserved.

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