

HOMEDICS®

LEADERS IN HEALTH & WELLBEING

✓ Register your product today at
www.homedics.co.uk/product-registration



MASSAGING BOOTIES

instruction manual

MS-10TNSM-EU
MS-10TNML-EU

3 YEAR guarantee

PRODUCT FEATURES:



ON/OFF BUTTON

FITTING THE BATTERIES

1. We recommend using Alkaline type batteries (4 x AA).
2. Open the zipper located at the back of each slipper.
3. Insert 2 x AA batteries into the battery compartment of each slipper.
Proper battery orientation is noted inside the battery compartment housing.
4. Put the battery compartment back in place and close the zipper.

INSTRUCTIONS FOR USE

1. Place your feet inside the slippers.
2. The massaging function is activated by pressing the On/Off button on the side of each slipper.
3. To turn off the massaging function simply press the On/Off button once again.

The Massaging Slippers are intended to provide a soothing massage to muscles – DO NOT use as a substitute for medical attention.

MAINTENANCE:

Cleaning

The Slippers are wipe clean only using a slightly damp, not wet, cloth. DO NOT immerse in water.
DO NOT use petroleum based or flammable cleaning agents because of fire risk.
DO NOT use abrasive cleaning agents.

Storage

- Remove batteries if equipment is not going to be used for extended periods of time.
- Switch off the product when not in use.
- Place it in a safe, dry and dust-free place.

FKA Brands Ltd

Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
Customer Support: +44(0) 1732 378557 | support@homedics.co.uk



IB-MS10TN-0518-01