






# SHIATSUNECK MASSAGER WITH HEAT

instruction manual



NMS-255-GB

# **PRODUCT FEATURES:**

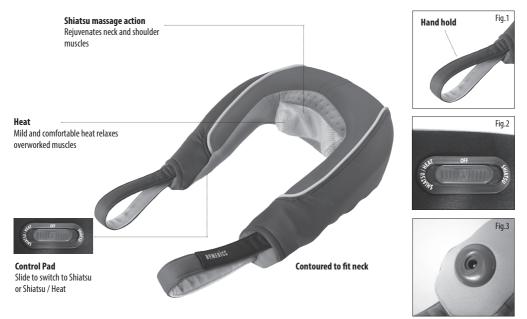

# **INSTRUCTIONS FOR USE:**

### Adaptor Instructions

- 1. Insert adaptor jack into the plug underneath left handle strap. (Fig. 1)
- 2. Plug into a AC 220v outlet.
- This unit incorporates a wire frame allowing you to contour the unit to your neck. Place massager on your neck and mold the wire framing to desired position.
- To activate the massage action, press the power button on the control pad. (Fig. 2)
- Press once for low speed, a second time for high, and press again to turn the unit off.
- To activate the heat function, with massage running, press the heat button on the control pad. (Fig. 2).
- 7. To turn off the heat function, simply press the heat button again.

#### NOTE: Heat is a mild, comfortable heat.

(-

NOTE: There is a 15 minute auto shut-off on this appliance for your safety. This feature should not be considered as a substitute for 'OFF'. Always remember to switch the appliance off when not in use.

## MAINTENANCE:

#### Cleaning

Unplug the appliance and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.

- Never allow water or any other liquids to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

## To wash cover

- Unzip and remove cover.
- · Follow instructions on wash label.
- Once dry, take care refitting cover and then zip up.



#### Storage

Place the appliance in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, DO NOT wrap the power cord around the appliance. DO NOT hang the unit by the cord.

FKA Brands Ltd Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK Customer Support: +44(0) 1732 378557 | support@homedics.co.uk