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**HoMedics  
Service Center, Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI  
48390**

**correo electrónico:  
[cservice@homedics.com](mailto:cservice@homedics.com)**

# NeckPleaser™

## Cordless Neck Massager

**Instruction Manual and  
Warranty Information**

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
**1** year  
limited warranty

## IMPORTANT SAFEGUARDS:

**Caution: Please read all instructions carefully before operating.**

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- Do not place or store appliance where it can fall or be dropped into a tub or sink.
- Do not place or drop into water or any other liquid.
- This unit should not be used by children without adult supervision.
- Turn off appliance when not in use.
- Never operate this appliance if it has been damaged or if it is not working properly. Return it to the HoMedics Service Center for examination and repair.
- Keep away from heated surfaces.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- This appliance is designed for household use only.

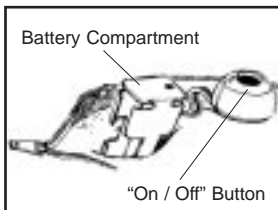Figure 1

## SAVE THESE INSTRUCTIONS Instructions for Use

- To install batteries, open zipper located along back side of cushion.
- Pull out compartment from foam padding.
- Place two "AA" size batteries (not included) into battery compartment and close zipper. (The Compartment indicates the proper position of batteries).
- The On/Off button can be operated without opening zipper. Simply, locate button and press down through the cloth cover.
- Place behind your head to secure around your neck. Never fasten Neck Pleaser too tight.

**Caution: All servicing of this massager must be performed by authorized HoMedics Service Personnel only.**

## Maintenance

### TO CLEAN:

Clean only with a soft, slightly damp cloth. Never allow water or any other liquids to come into contact with the unit.

- Do not immerse in any liquid to clean. To clean the fabric cover, wipe with damp sponge, using a mild liquid detergent or suds. Wipe surface with a clean, dry cloth.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish or paint thinner to clean the massager.
- Do not attempt to repair this unit. There are no user serviceable parts. For service, send the unit to the HoMedics address listed in the warranty section.

### TO STORE:

Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or snag the fabric surface.

**Soothing Relief**  
Provides a soothing massage to help relieve stress

**Portable Design**  
Easy to use anywhere

**One Size Fits All**

**Cordless Operation**  
Convenient battery operation (2 "AA" batteries not included)

**Luxurious, Soft Cover**  
Plush fabric feels soft to the touch

**Note: Fabric color may vary from the white fleece shown here**

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of one year from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of one year from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

**For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)**

**Mail To:**

**HoMedics  
Service Center, Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI  
48390**

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