ParaSpa[™]Plus

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(Válida únicamente en los EE.UU.)

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**Instruction Manual and
Warranty Information**

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Enjoy warm, soothing relief from muscle tension, and moisturize dry hands.

Thank you for purchasing the HoMedics ParaSpaPlus refreshing home paraffin bath. This, like the entire HoMedics line, is built with high-quality craftsmanship. We hope that you'll find it to be the finest product of its kind.

The ParaSpaPlus paraffin bath turns your home into a salon, providing you with the ultimate experience by helping you to relax, relieve tension and soothe sore muscles.

Relax. Renew. Simplify.

IMPORTANT SAFEGUARDS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.
DANGER—TO REDUCE THE RISK OF ELECTROCUTION:

- Do not reach for an appliance that has fallen into water. Unplug it immediately.
- Do not place appliance where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid. Store in a cool, dry place.
- Always unplug this appliance from the electrical outlet before cleaning.
- Do not plug or unplug this product when hands are wet.
- Do not use while bathing.

WARNING - TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- Close supervision is necessary when this appliance is used by, on, or near children, invalids or disabled persons.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- Use this appliance only for its intended use as described in this manual.

This unit is designed for household use only. This unit is not designed for commercial use. This unit is not a professional device.

- Do not use attachments not recommended by HoMedics, specifically any attachment not provided with the unit.
- Do not operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to the HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.
- Do not drop or insert any object into any opening.
- Do not carry this appliance by the supply cord or use the cord as a handle.
- Do not use outdoors.
- Do not let cord hang over the edge of table or counter where it can be pulled on by children or tipped over unintentionally.
- Do not place on or near any hot gas, flames, electric element, or in a heated oven.
- Do not operate where aerosol spray products are being used or where oxygen is being administered.
- Never use this product while sleeping or drowsy.
- Never use sharp instruments to remove solidified paraffin wax from the unit.
- Use appliance on a sturdy level surface only.

SAVE THESE INSTRUCTIONS

Caution: Please read all instructions carefully before operating.

SPECIAL WARNINGS:

IF YOU HAVE OR HAVE HAD ANY SKIN CONDITIONS, DISEASES AND OR HEALTH PROBLEMS, YOU SHOULD CONSULT YOUR PHYSICIAN BEFORE USE.

- Do not apply paraffin wax on open cuts or wounds, warts, inflamed skin, or when the sensation of any extremities is decreased.
- Consult your physician if you have any questions or concerns about acute inflammation and peripheral vascular disease or decreased sensation of any extremity before using.
- Do not apply the hot wax near eyes or other heat sensitive areas.
- Discontinue use and consult a physician if the paraffin wax feels too hot or if it does not feel warm enough. This may indicate a possible physical problem.
- Discontinue use and consult your physician if dermatitis, discomfort, irritation, or pain occurs from continued paraffin treatments.
- Test the paraffin wax on the inside of your wrist to make sure the temperature is comfortable.

- Do not overfill the unit and always keep the paraffin wax level about 1.5 inches from the top of the bath.
- Never operate the unit without paraffin wax.
- Do not heat paraffin wax in its plastic packaging, nor heat it over an open flame, on a range, stove or in a microwave.
- Do not pour the melted paraffin wax from the tub/unit.
- Unplug unit before filling with paraffin wax.
- Do not use accessories or wax not recommended or sold by HoMedics.
- If the supply cord is damaged, the entire unit must be replaced by the HoMedics Service Center.
- Do not put water into the tub/unit.


Caution:

This appliance has a polarized plug (one blade is wider than the other). To reduce the risk of electric shock, the plug is intended to fit into a polarized outlet only one way. If the plug does not fit into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician.

Therapeutic Benefits

- Paraffin heat therapy seals in heat and contours to the afflicted area for the ultimate moist heat treatment.
- Useful in symptomatic relief of pain caused by arthritis, bursitis, and chronic joint inflammation.
- Relaxes muscles, relieves stiffness and muscle spasms.
- Stimulates circulation.
- Softens and soothes dry, cracked skin.

Figure 1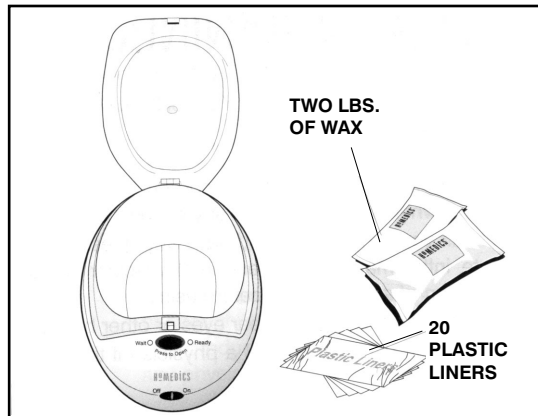


Important Note:

This unit may feel very warm to sensitive skin. However, in order to fully experience the therapeutic benefits of paraffin heat therapy, the wax must be heated to a temperature between 123-131 degrees Fahrenheit. This is the temperature utilized by the leading hospital brand paraffin baths.

For proper application, quickly dip your hand into the melted paraffin wax and remove immediately. This should be repeated 6-10 times. Never soak in the wax for more than 1-2 seconds.

If you are very sensitive to heat or feel that this appliance may not be for you, please return this product to the place of purchase, prior to use.

Figure 2

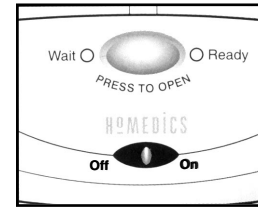How to Set Up the ParaSpaPlus Unit

1. Remove all contents from packaging and check to make sure you have all of the following components (See Figure 1) of the ParaSpa paraffin bath:
 - Paraffin bath tub
 - 2 lbs. of Paraffin Wax
 - 20 plastic liners
2. Place unit on a level and sturdy table or surface and lift the lid.
3. Make sure ParaSpa tub is empty and clean before putting wax into tub.
4. Open the bags of paraffin wax and place wax in the tub. The unit is designed to hold a maximum of **2.5 lbs.** of paraffin wax and a minimum of 2 lbs. Always keep the level of paraffin wax filled to the "MIN" level marked on the inside of the tub but not over the "MAX" line.
5. Plug the unit into a 110V/120V A/C household outlet.
6. Move the blue button to the "ON" position. The large oval button will glow red. The red light indicates that the melting cycle has started and is not yet ready for use. See Fig. 2. (Approximate melting time is 2 hours).
7. The oval button will change from red to green when the wax is ready to use. Depress the oval button to open the lid. Never dip your hand in the wax until the button has changed from red to green.
8. Move the blue button to the "OFF" position when finished using the unit.

NOTE: NEVER dip your hand into the ParaSpa bath while the LED light is red. This is the melt mode and will be too hot.

ParaSpaPlus™

Figure 3

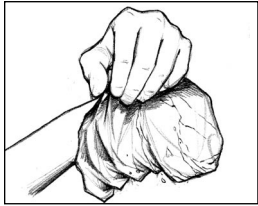
Figure 4

How to Use Paraffin Treatment

1. Before immersing your hand, test the paraffin wax on the inside of your wrist to make sure the temperature of the wax is comfortable. Remove all jewelry before dipping (i.e. rings, bracelets, watch). **NOTE:** It is suggested you wash the area to be treated and apply lanolin-based hand lotion over the area before dipping it into the paraffin wax. This will allow the wax to be removed more easily and will help moisturize the skin.
2. Dip your hand into the melted paraffin wax and remove immediately. Wait a few seconds until the wax solidifies. Try to avoid moving your hand while wax is applied. **NOTE:** Dipping process is quick. Do not leave hand in the tub. Do not soak. Dip quickly in and out.
3. Repeat this step several times until several layers have formed, 6-10 coatings are suggested.
4. For longer heat treatment, place a plastic liner (provided) over the treated area and twist bag at opening. Keep paraffin wax and liner on treated area for 15-30 minutes for ultimate therapy. (See Figure 3)
5. Peel off the wax from the treated area. If using a plastic liner, peel the plastic lining from the area being treated. Paraffin wax will peel off with liner. (See Figure 4)
6. Discard used wax. If you are the only person using the ParaSpa, the wax can be re-used. Simply put it back into the tub to re-melt.

Maintenance

Periodically, the used portion of paraffin wax should be replaced and the unit should be cleaned.

1. Unplug unit, allow paraffin wax to solidify overnight.
2. Push on one end of paraffin wax block. This will tip it up. Lift the block out of the unit and dispose of it. **DO NOT USE SHARP OBJECTS TO REMOVE THE PARAFFIN WAX BLOCK, THIS MAY SCRATCH OR DAMAGE THE TUB.** If the paraffin wax does not remove easily, plug unit in and allow to heat for 10-15 minutes then remove the paraffin wax block and unplug the unit.
3. Wipe the inside of the unit with a disposable towel to remove residual dirt and paraffin.
4. With a damp, soapy cloth, wipe clean. **DO NOT POUR WATER INTO THE UNIT, OR IMMERSE THE UNIT IN WATER.**
5. Store in a cool, dry place. When you are ready to re-use the ParaSpa bath for the next heat treatment, repeat steps 2-7, "How to set up the ParaSpaPlus unit". Then, follow steps 1-6, "How to Use Paraffin Treatment".

ONE YEAR LIMITED WARRANTY

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of one year from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of one year from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit:
www.homedics.com

©2002 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics® is a registered trademark of HoMedics Inc. and its affiliated companies. ParaSpa™ and Body Basics™ are trademarks of HoMedics Inc. and its affiliated companies.

All rights reserved.
IBPAR70-A

Mail To:
HoMedics
Service Center Dept. 168
3000 Pontiac Trail
Commerce Township, MI
48390

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