

ONE YEAR LIMITED WARRANTY

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of one year from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of one year from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit:
www.homedics.com

Mail To:
HoMedics
Service Center, Dept. 168
3000 Pontiac Trail
Commerce Township, MI 48390

e-mail: cservice@homedics.com

© 2002 HoMedics, Inc. and its affiliated companies, all rights reserved.
HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies. StressEase™ and The beauty of living well.™ are trademarks of HoMedics Inc. and its affiliated companies. All rights reserved.
IB-PM4C



StressEase™ Roller Massager

IMPORTANT SAFETY INSTRUCTIONS:

Caution: Please read all instructions carefully before operating.

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- DO NOT place or store appliance where it can fall or be dropped into a tub or sink.
- DO NOT place or drop into water or any other liquid.
- This unit should not be used by children without adult supervision.
- Turn off appliance when not in use.
- NEVER operate this appliance if it has been damaged or if it is not working properly. Return it to the HoMedics Service Center for examination and repair.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. DO NOT use this product as a substitute for medical attention.
- This massager should not be used over swollen or inflamed areas or skin eruptions.
- This appliance is designed for household use only.
- This appliance is designed for personal, non-professional use only. This appliance should be used only for personal, non-professional use.

SAVE THESE INSTRUCTIONS Operating Instructions

- To install batteries, open the battery compartment and place 2 "AA" batteries (not included) into the chamber as directed on the bottom of the compartment. Replace the cover.
- To turn unit on, press the power button located on the side of the massager.
- Roll the massager over the desired area lightly in a back and forth motion.
- After completing massage, turn the unit off by depressing the power button once.