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GB

Thank you for choosing HoMedics products. Please read these instructions for use carefully and keep them for further use.

1. Gently rub the back of your No Pain 3D to remove any adhesive residue left from the previous gel pad.
2. Remove the Self-Adhesive Gel Pad from its wrapping.
3. Peel the protective film off one side of the gel pads and place adhesive side onto the back of the device. Press firmly.
4. Carefully peel off the other protective film so that the adhesive gel pads are left in place.

## IMPORTANT SAFEGUARDS

- This Self-Adhesive Gel Pad is designed for the HoMedics No Pain 3D only. Only use No Pain 3D Self-Adhesive Gel Pads with your device. This product is not suitable for use with any other devices.
- We recommend that you replace the gel pads after 20 treatments to maintain optimum performance.
- Do not apply the gel pad to swollen or inflamed areas, open wounds, irritated or damaged skin.
- Cover the gel pad with the supplied protective film after every treatment. To maintain performance, do not clean the gel pads with hands, cloth, facial tissue or any fabric products.
- If the unit is causing discomfort, stop using immediately.
- Store gel pads in a cool, well ventilated place. Keep away from direct sunlight and high humidity.

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