HOMEDICS

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(Válida únicamente en los EE.UU.)

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**HoMedics
Service Center, Dept. 168
3000 Pontiac Trail
Commerce Township, MI
48390**

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The beauty of living well.™

TherapistSelect™ Wave Action™ Massager

**Instruction Manual and
Warranty Information**

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2 year
limited warranty

**IMPORTANT SAFEGUARDS
WHEN USING ELECTRICAL PRODUCTS,
ESPECIALLY WHEN CHILDREN ARE PRESENT,
BASIC SAFETY PRECAUTIONS SHOULD
ALWAYS BE FOLLOWED, INCLUDING THE
FOLLOWING:**

**READ ALL INSTRUCTIONS BEFORE USING
DANGER - TO REDUCE THE RISK OF
ELECTROCUTION:**

- ALWAYS unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or showering.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink. DO NOT place in or drop into water or other liquid.

SAVE THESE INSTRUCTIONS

**WARNING - TO REDUCE THE RISK OF
BURNS, ELECTROCUTION, FIRE OR INJURY
TO PERSONS:**

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.

**Caution:
In case of
pregnancy,
diabetes or
illness, consult
your doctor
before using
massager**

- Use heated surfaces carefully. May cause serious burns. DO NOT use over sensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- Do not use outdoors.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.
- NEVER block the air openings of the appliance or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep air openings free of lint, hair, etc.
- NEVER use while sleeping or fall asleep while using the massager.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT use on or near eyes or other highly sensitive areas.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by supply cord or use cord as handle.
- DO NOT use the massager directly on one concentrated area of the body. Massagers are designed to be used in a continuous motion, constantly changing the massage area.
- To disconnect, turn all controls to the "OFF" position, then remove plug from outlet.
- This appliance is designed for household use only.

**Caution – Please read all instructions
carefully before operating.**

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.

- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- This unit should not be used by children or invalids without adult supervision.
- This product should never be used by any individual suffering from any ailment that would limit or altogether inhibit the user's capacity to feel or have any sensation in the body part that the product is being used on.

Instructions for Use

To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. DO NOT change the plug in any way.

1. ALWAYS make sure the unit is set to the "OFF" position before inserting the plug into a 120 volt electrical outlet.
2. **Message:** To turn the unit on, simply slide the ON/OFF switch up located on the right side of the unit. The corresponding LED light will confirm your selection. To adjust the intensity of the massage, slide the variable switch down for lower intensity and up for higher intensity. (Fig. A)

**Caution:
Never use
the heat
feature
without the
proper
attachment.**

3. **Message with Thermal Heat:** Viewing the bottom side of the unit simply twist the massage head counter-clockwise (to the left) and pull the attachment head off (Fig. B). Locate the two red (heat) attachments and place them onto the post and perpendicular to the massage unit. Once on the post, twist the attachment clock wise (to the right) to secure. The heads should now be aligned in the same direction as the unit (Fig. C). Slide the ON/OFF switch up and then slide the red switch up to activate the heat function. The corresponding LED light will confirm your selection. Allow a few minutes for heat to fully activate.
NOTE: To eliminate the risk of being burned, never use the heat function without the attachments. Do not use the thermal heat continuously for more than 15 minutes. Do not use if you have sensitive skin, poor circulation, discolored areas, swelling, burned, inflamed or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.
4. Apply massager head lightly to the area you wish to treat.
NOTE: Do not use the massager on one concentrated area of the body. This massager is designed to be used with continuous motion, constantly changing the massage area so it is not used on one concentrated area of the body.
CAUTION - as stated in the "IMPORTANT SAFEGUARDS" section of this manual, never use this appliance on or near eyes or other highly sensitive areas.
5. Operation of longer than 15 minutes is not recommended and may cause overheating.
6. When treatment is completed, turn unit off, allow to cool, unplug the power supply and store properly in a dry place.

Figure A

Figure B

Figure C

To Change Attachments

Your massager comes equipped with four custom massage head attachments. Each attachment is designed to provide a different massage sensation:

- Large accunode massage heads rejuvenate worn muscles
- Micro finger massage heads invigorate tense muscles
- Rolling massage heads for a gentle massage action
- Heated massage heads offer soothing heat therapy

Viewing the bottom side of the unit, simply twist the massage head counter-clockwise (to the left) and pull the attachment head off (Fig. B). To apply massage head, place the new attachment onto the post and perpendicular to the massage unit. Once seated onto the post, twist the attachment clockwise (to the right) to secure. The heads should now be aligned in the same direction as the unit (Fig. C).

NOTE: Micro finger massage heads were designed to provide a vigorous massage. This massager should be used with continuous motion, constantly changing the massage area so it is not on one concentrated area of the body. We recommend that you place a thin layer of clothing or a sheet between the micro finger massage heads and the areas of application.

Maintenance

TO CLEAN:

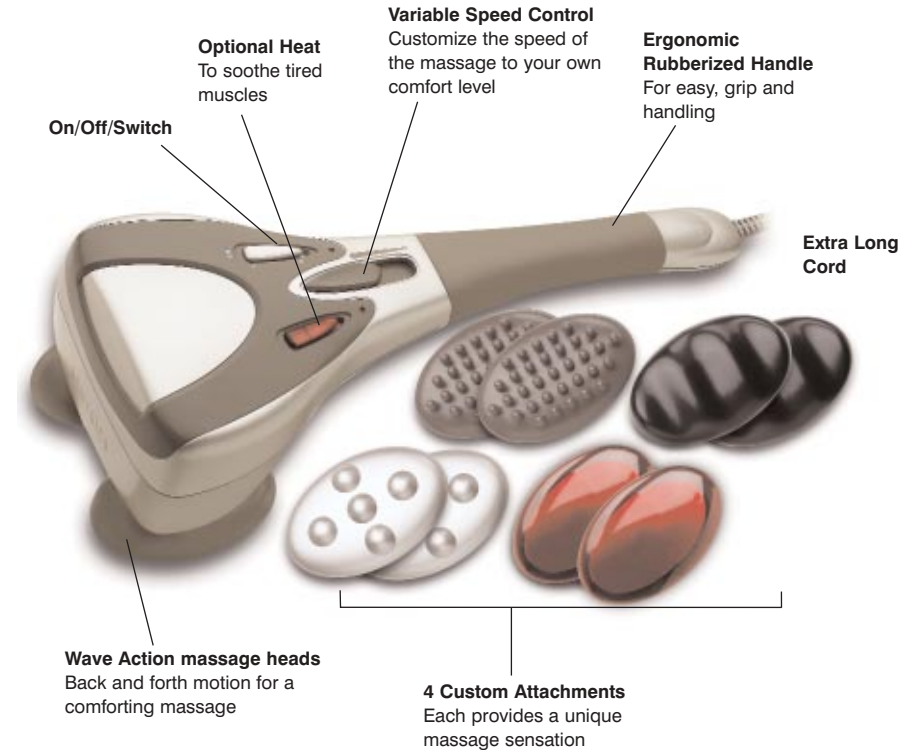
Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe. NEVER allow water or any liquids to come into contact with the handle.

TO STORE:

Unplug the appliance from the outlet and allow it to cool before storing in its box or a clean, dry place. Coil the cord and secure it with the supplied fastener. DO NOT hang the unit by the powercord.

Caution:
All servicing of this unit must be performed by authorized HoMedics service personnel only.

TherapistSelect Wave Action Massager

HOMEDICS

TWO YEAR LIMITED WARRANTY

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

Mail To:

HoMedics
Service Center, Dept. 168
3000 Pontiac Trail
Commerce Township, MI
48390

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WV-100H

©2002 HoMedics, Inc. and its affiliated companies, all rights reserved.
HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies.
Therapist Select™, WaveAction™ and The beauty of living well.™ are trademarks of
HoMedics, Inc. and its affiliated companies. All rights reserved.

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